

STEP 1

DOWNLOAD THE LUCID HEARING APP®



Scan this code with your phone camera to download the app




STEP 2

CREATE YOUR LUCID HEARING® ACCOUNT








Open the app and set-up your account

Your first screen will be a welcome screen. Click on **Start Using Lucid Hearing** to begin the sign-in process.

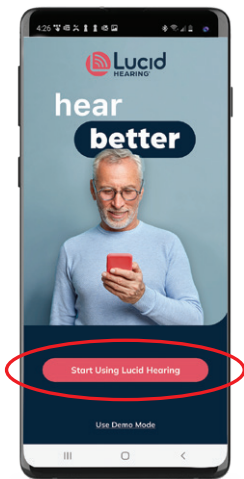

Follow the on-screen instructions to login or create an account. **Be sure to verify your email to complete the process.**

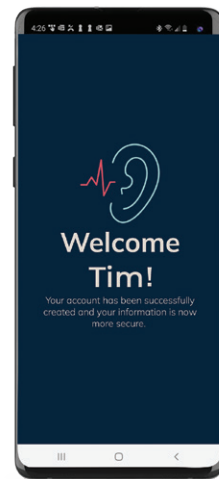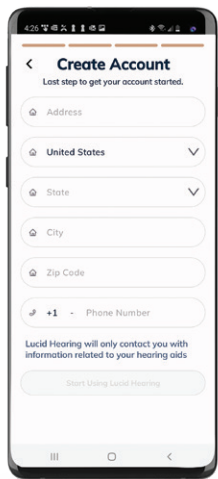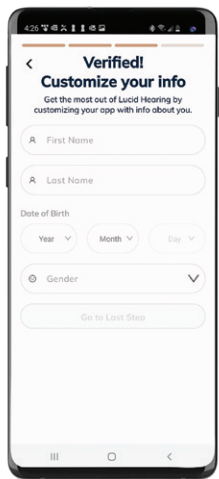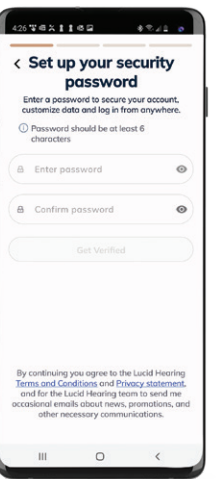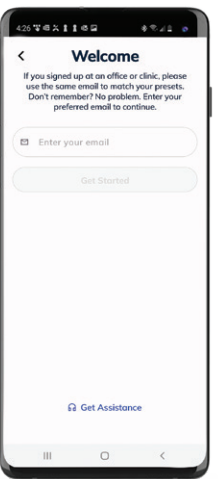


Once set-up is complete, you'll be greeted by a welcome screen.


STEP 3

CONNECT/PAIR YOUR HEARING AIDS TO THE LUCID HEARING APP


Your hearing aids should be charged and taken out of their case to power on.

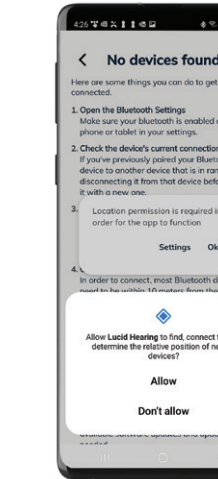

Accept app permissions to find your device.

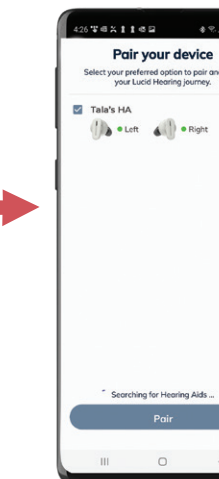


Your hearing aids will appear. Click **"Pair"** both times it appears. Your hearing aids are now paired.

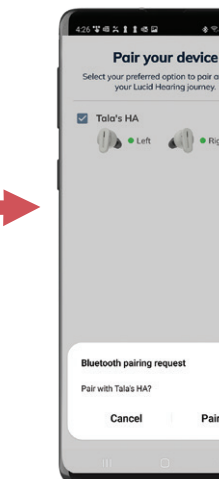

STEP 4

FIND YOUR PERFECT FIT

Attach a set of ear tips to the device and experiment with different ones. The tip should fit snugly enough to seal the ear but not be uncomfortably tight.

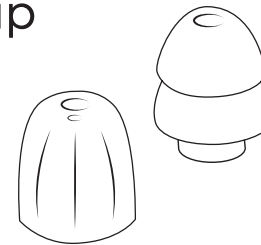


STEP 5

ADJUST THE VOLUME OR PERSONALIZE YOUR SOUND

You can adjust:

- Volume
- Programs
- Audio EQ
- LucidShape®

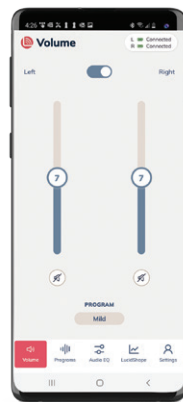

STEP 6


ENJOY BETTER HEARING!

Go connect to the world you love.

©2023 Lucid Hearing Holding Company, LLC. Lucid Hearing and Lucid Hearing App are registered trademarks of Lucid Hearing Holding Company, LLC. Tala is a trademark of Lucid Hearing Holding Company, LLC. Android is a trademark of Google LLC. Google Play and the Google Play logo are trademarks of Google LLC. All rights reserved.

Need additional help?

Refer to the **User Manual** or our online video tutorials.

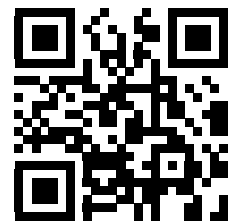

Scan the code for video tutorials

or visit

lucidhearing.com/support-video

WE'RE HERE TO HELP!

CALL TOLL FREE:
(800) 785-6052

Monday - Saturday
8AM – 6PM Central Time

EMAIL:

support@lucidhearing.com

IFU-1066 Rev01

TALA™ QUICK START GUIDE

android 

**Compatible
Device Required**



Scan with your phone camera to check your device compatibility

(800) 785-6052